



 **Artimax™**

America's #1 selling Dental articulation tray system

Quadrant Tray



*Follow the proven success for
much reduced chairtime/remakes
with no extra cost*

Benefits

- 3 different types of trays customized to your die stone linear expansion rate
- Polymer base... Zero expansion model base: Far less interproximal contact problems & High Occlusion problems
- Superior accuracy
- Fast, Simple, Easy, Efficient & Versatile
- Patented pin hole covering methods


U.S. Patent Nos.: 7,147,465, 6,948,932 and Patents Pending

100's of Die Stones.
One complete tray system
for One you...



www.ArtimaxUSA.com

Customized for YOU... 3 types* of trays

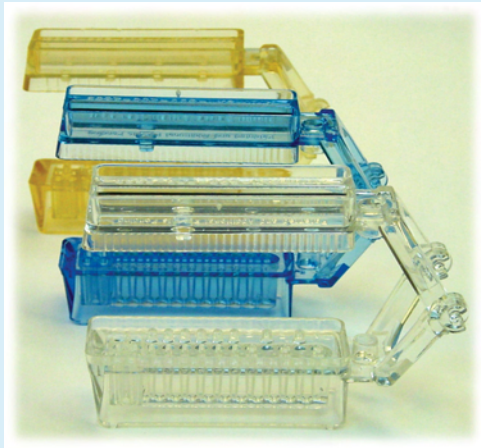


Choose your version based on...

- your die stone expansion rate
- ceramists' preference
- model technician's preference
- whether you have contact problems

Choose whatever type of stone best suits your lab and Artimax will do the rest. With 3 choices of trays you are sure to find complete compatibility.

Articulator Tray	Die Stone Expansion			Main benefits
	Low	Med	High	
Wall type	●	●		Maximum retention & die stability Comparatively easier for beginners
Wall-less 1	●	●	●	Easier confirmation of complete die seating Recommended for high expansion stones
Wall-less 2	●	●	●	Recommended for labs experiencing contact problems consistently

* All 3 types of articulators are currently available in clear and blue

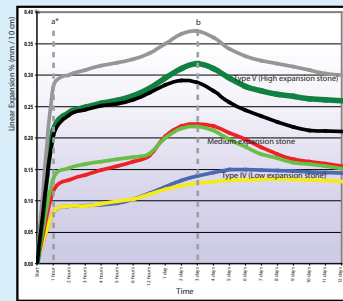

Stone expansion test results for some of the leading stone brands in the market



See our stone expansion test results and its effect on contact problems and occlusion problems on our web site

Technique Guide

Takes only 5 minutes per case. Everything you need is already built into Artimax articulator



Pour the opposing side first. This will help stabilize the impression position on the tray.



Mark the center of the prepped tooth on the impression and then on the working tray.



Insert the pin into the marked pin holes, puncturing the membrane that covers the pin holes



Pour the die stone onto the working tray and invert over the impression. Let the stone set.



Finished model in 5 minutes.

- 123456 Quadrant manual (1 page - simplified)
- 123456 Quadrant manual (8 pages - detailed, step-by-step)



Download manual on our web site

Ordering Information

- 123456 Quadrant Trial Kit (Small)**
10 Wall version Quad trays, 10 Wall-less Quad trays
- 123456 Quadrant Trial Kit (Medium)**
20 Wall version Quad trays, 20 Wall-less Quad trays
- 123456 Quadrant Trial Kit (Large)**
50 Wall version Quad trays, 50 Wall-less Quad trays
- 123456 Quadrant Wall version (Clear 123456 C/ Blue 123456B)**
50 Wall version Quad trays, 200 medium size brass dowel pins, 10 posterior stop rods
- 123456 Quadrant Wall-less version 1 (Clear 123456 C/ Blue 123456 B)**
50 Wall-less version Quad trays, 200 medium size brass dowel pins, 10 posterior stop rods
- 123456 Quadrant Wall-less version 2 (Clear 123456 C/ Blue 123456B)**
50 Wall-less version Quad trays, 200 medium size brass dowel pins, 10 posterior stop rods



Our Guarantee - If you are not 100 % satisfied, simply return the product for a complete refund. It's that easy.